V2.7

fluorecare®
SARS-CoV-2 Antigen Test Kit Test Procedures
(Colloidal Gold Chromatographic Immunoassay)
Product code:MF-68

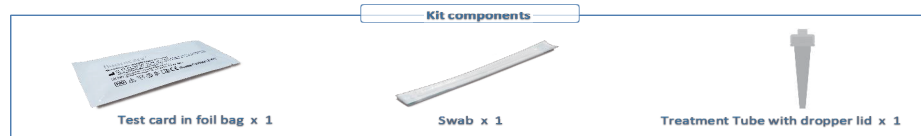
1 Nasal swab

Sampling illustration

1.1 Wash or disinfect your hands before using the kit. Make sure they are dry before starting.

1.2 Confirm the test components

1.3 Tear the aluminum foil bag and take out the test card. Place the test card on a flat surface, such as a table. Avoid touching the result and sample hole areas.

1.4 Sampling illustration (nasal swab)

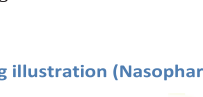
1 Carefully remove sterile nasal swab from the packaging. (Avoid touching the end with the cotton swab)


2 Insert the nasal swab into the left nostril to a depth of 2.5 cm (1 inch) from the edge of the nostril.


3 Rotate the nasal swab on the nostril wall (mucous membrane) 5 times to ensure adequate sampling.

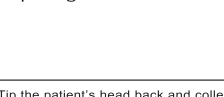4 Repeat the process in the right nostril with the same nasal swab, collecting from both nasal passages to ensure an adequate sample.

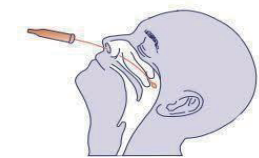1.5 Sampling illustration (Nasopharyngeal swab)

1. Tip the patient's head back and collect sample from the nostril that has more mucus (head should be inclined from vertical for proper specimen collection).
2. Insert the swab through the nostril entry and then slowly move along the bottom of the nasal cavity (Move gently to avoid traumatic bleeding).
3. When the tip of the swab reaches the posterior wall of the nasopharyngeal cavity, gently rotate it several times. (Collect as much secretion as possible)
4. To prevent reflex coughing, stop for one minute.
5. Slowly remove the swab.
6. Test the sample as soon as possible.

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
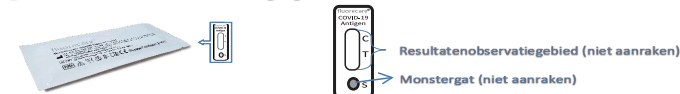
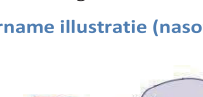
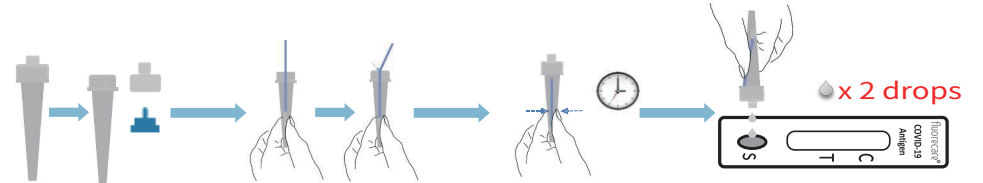
fluorecare®

Sample Applying

- 1 Open the treatment tube dropper lid. And remove the inner plug.
- 2 Place the swab sample in the tube, then break the swab at the swab node, and leave the lower half in the treatment tube.
- 3 Load the dropper lid, squeeze the swab in the tube for 10 times. Then wait for 5 minutes of sample reaction.
- 4 Open the lid of the dropper and vertically add 2 drops (about 60µL) of sample solution to the sample hole. Wait for 15 minutes of reaction at room temperature.

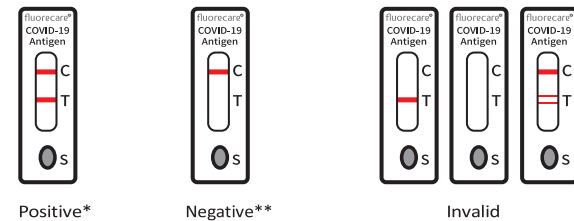2 Read the result

*If your test result is positive, please follow the local government protocols for your country, ensuring that you isolate yourself to minimise the spread of your infection to others

**If your test is negative, you should continue to follow the current coronavirus measures for your territory. Negative test results do not rule out the presence of coronaviruses in the sample at any time because they may be present below the minimum detection limit.

NOTE: After the observation, the kit including the sample treatment tube is put back into the bag and thrown into the trash can.

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